


MYCOFAST[®] *Revolution* ATB+

UROGENITAL MYCOPLASMA DIAGNOSIS SCREENING METHOD

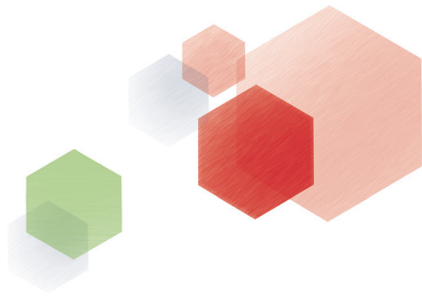

Expanding the scope of treatment options!

Detection, Enumeration, Identification and Antimicrobial Susceptibility Testing

- ◆ Extended panel of antibiotics tested, in full compliance with worldwide recommendations
- ◆ Complete diagnosis in a two-phase approach
- ◆ Cost-effective screening in 24 hours
- ◆ The right pathologic threshold tested for each specimen type
- ◆ Visual, colorimetric reading

BENEFITS

Complete diagnosis

STEP 1 (MYCOFAST® Screening *RevolutioN* tray) - 24h*

- Detection of *U. urealyticum* (U.u.) and *M. hominis* (M.h.)

STEP 2 (MYCOFAST® *RevolutioN* ATB+ tray) - 24h*

- Enumeration, Identification
- Antimicrobial Susceptibility Testing

Cutting-edge performance

Sensitivity

- Detection and enumeration of *U. urealyticum* at 10^3 and $\geq 10^4$ CCU/mL
- Detection and enumeration of *M. hominis* at 10^4 CCU/mL
- The right pathologic threshold tested for each specimen type

Specificity

- **MYCOFAST® Screening *RevolutioN*:**
Specific M.h. and U.u. growth media: more reliable results, reduced number of erroneous results
- **MYCOFAST® *RevolutioN* ATB+:**
Identification secured by checking U.u./M.h natural resistance to antibiotics (Clindamycin and Erythromycin)

Reliability

- 6 antibiotics (TEL/LVX/MXF/ERY/CLI/TET) tested in compliance with CLSI guidelines (Clinical & Laboratory Standards Institute)
- 5 other antibiotics tested (MIN/OFX/ROX/PRI/JOS)

Fast results

- Integrated growth activator for *M. hominis*


Ease of use

- Ready-to-use reagents
- Each well pre-filled with growth medium
- Visual colorimetric reading
- No instrument required

Flexibility and Safety

- Optimized transport medium: Specimens preserved up to 72 hours before analysis
- Clip-on closing system for the *RevolutioN* ATB+ tray

METHODOLOGY

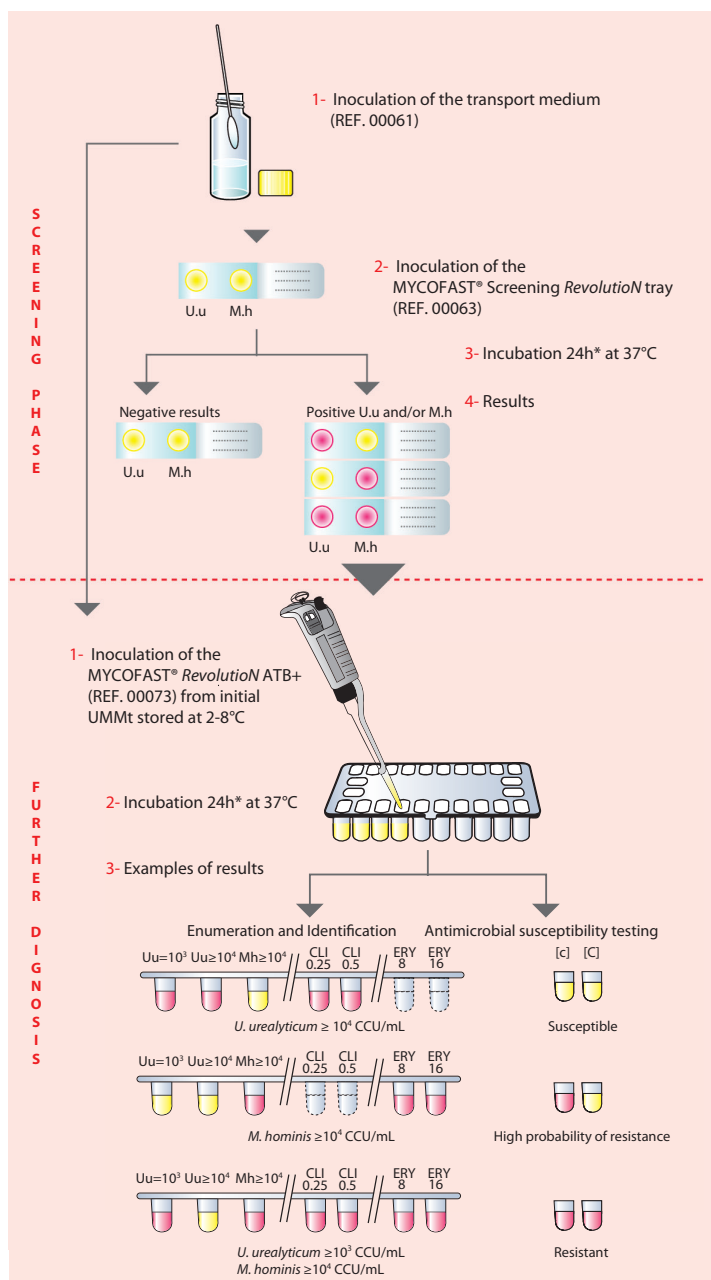

* Reading of all samples in 24h.
Second reading needed only for liquid samples having negative results within 24h.

PRODUCT RANGE

REF.	NAME	QTY	PURPOSE
00061	UMMt <i>RevolutioN</i>	50 vials	Transport & Inoculation medium
00063	MYCOFAST® Screening <i>RevolutioN</i>	50 tests	Screening tray for detection of Uu/Mh
00073	Complement MYCOFAST® <i>RevolutioN</i> ATB+	25 tests	Complete tray for further diagnosis
00064	MYCOPLASMA Stabilizer (optional)	100 tests	Additive for sample storage > 56 hours
00900	MYCOPLASMA Control (optional)	12 tests	U.u. and M.h. control strains

Product not available in the U.S
Please contact your sales representative for product availability in your country

: ELITech Microbio (France)

WORLDWIDE OFFICES

Headquarters	T: +33 1 41 45 07 10		
Australia	T: +61 1800 815 098	Serbia	T: +381 11 2467119
Belgium & Luxembourg	T: +32 92 820 531	Switzerland	T: +41 26 663 86 60
Brazil	T: +55 27 3025 1415	The Netherlands	T: +31 313 430 581
France	T: +33 4 83 36 10 82	UK	T: +44 1442 869320
Italy	T: +39 02 48 40 35 42	United States	T: +1 609 216 7361
New Zealand	T: +64 0800 555 611		

ELITechGroup
SOLUTIONS
tailored to your needs
www.elitechgroup.com
info@elitechgroup.com