

DOR-BAR

GROUP A STREPTOCOCCUS
SERODIAGNOSIS



KIT FOR THE TITRATION OF ANTI-STREPTODORNASE B ANTIBODIES (ASDOR)

- ◆ Performant
- ◆ Simple and convenient
- ◆ Visual and colorimetric results

DOR-BAR

BENEFITS :

CONVENIENT AND COST-EFFECTIVE

- No instrument required
- Space-saving trays for the incubator

EASY TO USE - FAST

- Simple methodology
- Only one incubation step (4h at 37°C)
- Visual and colorimetric results

SAFE RESULTS

- Negative control C- (well 9 containing streptodornase B but no serum)
- Positive control C+ (well 10 containing serum but no streptodornase B)
- Standard control serum (Human serum in a separate vial)

Take the diagnosis further with the ASO-BAR kit (Ref. 04125)

- Titration of anti-streptolysine O antibodies (ASLO)

PRODUCTS :

DIL : 5 ml vial of diluant.

SUBS : Vial of lyophilized medium containing the substrate (DNA) and a colour indicator. To be reconstituted with 600 µl of sterile distilled water.


DOR-BAR tray : Divisible tray of 2 x 10 wells (2 tests), individually wrapped in aluminium pouches. The trays contain increasing concentrations of dehydrated streptodornase B.

Control Serum: 1 vial of 2.2mL positive control, standard at 600 U/mL (human serum).

METHODOLOGY :


1. Dilution of serum

Dilute the serum sample (1/80) with the diluant :
10 µL of serum
790 µL of DOR-BAR DIL

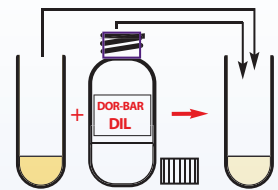

2. Distribution of serum into the tray


Dispense 50 µL of the serum dilution into each of the wells (except C-)

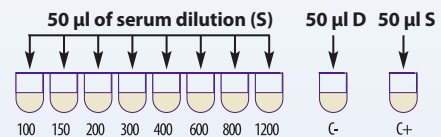

3. Regeneration of substrat

Transfer 600 µl of H₂O into SUBS


4. Distribution of substrat

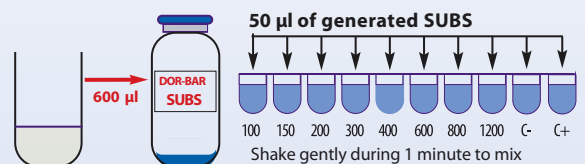

6. Results

5. Incubation

Incubate 4 hours at 37°C

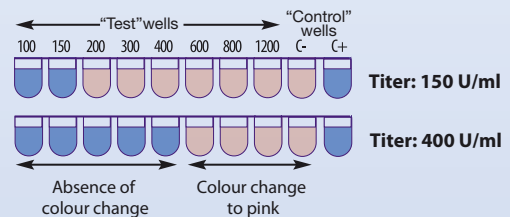

CATALOGUE :

REFERENCE	NAME	QTY
04135	DOR-BAR	Coffret de 12 tests

DISTRIBUTOR CONTACT INFORMATION:

Product not available in the US
Please contact your sales representative for product availability in your country

WORLDWIDE OFFICES:

Belgium T: +32 9 2820 531
Brazil T: +55 27 3025 1415
France T: +33 4 90 17 54 50
Italy T: +39 02 48 40 35 42

Middle East & Africa T: +971 4 375 2744
Serbia T: +381 11 2467119
Switzerland T: +41 26 663 86 60
The Netherlands T: +31 485 511 675

UK T: +44 1442 869320
United States T: 1 800 453 2725

Headquarters T: +33 1 41 45 07 13
www.elitechgroup.com
info@elitechgroup.com

