
Antimicrobial Susceptibility Testing of Urinary *Enterobacteriaceae*

*Manual, routine,
rapid test*

- Results within 4-5 hours
- Easy to use and cost-effective
- Visual, colorimetric reading
- "PRESTO ABG®" standardization

URIFAST®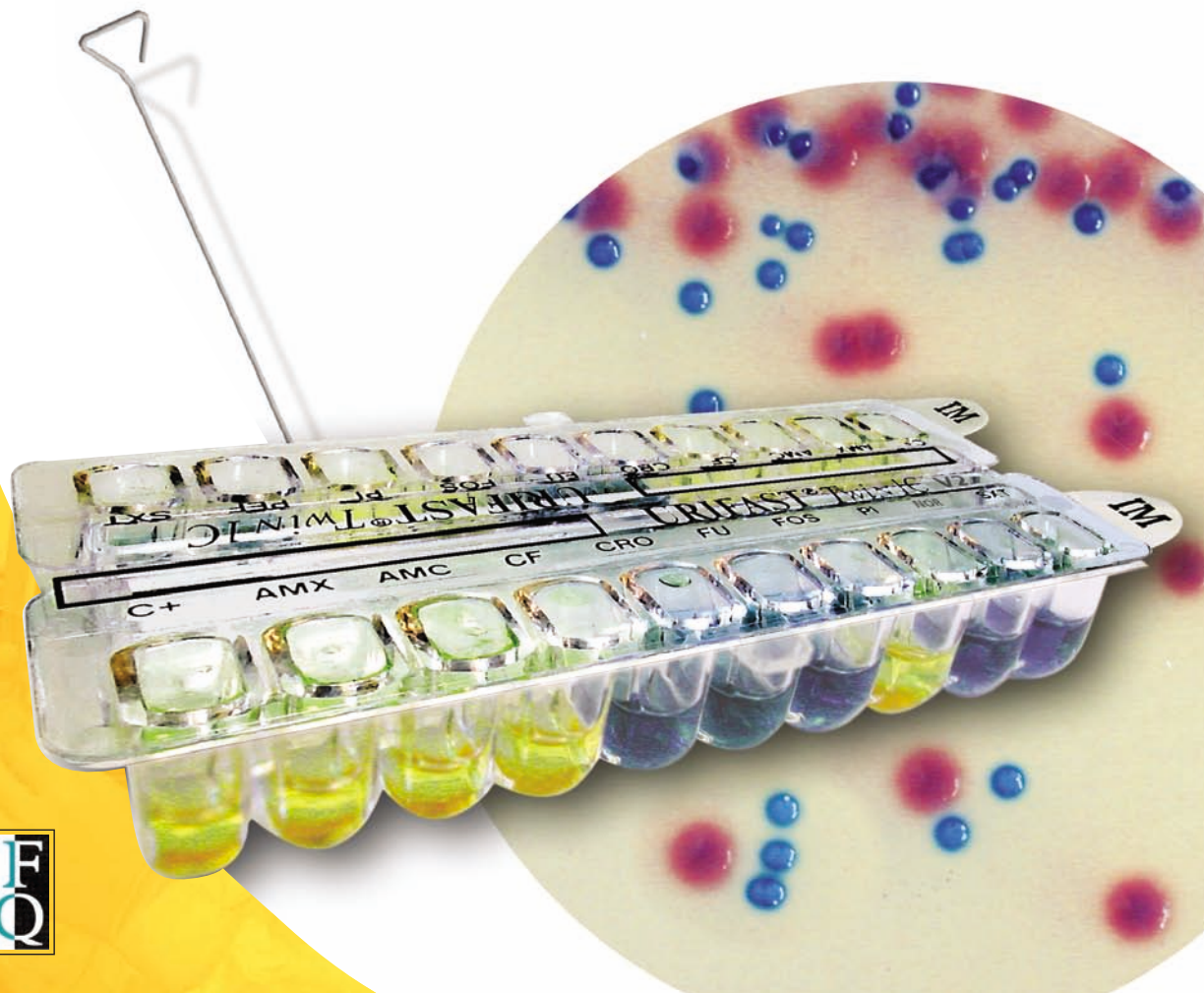

BENEFITS

Fast

- Handling time: 1 minute
- Results within 4-5 hours

Easy to use

- Standardization of the inoculum with the PRESTO ABG®:
 - No suspension medium
 - Direct inoculation of the M4H medium
 - One isolated colony required
- Reading by indicator color change

Cost-effective

- No instrument required:
 - No densitometer
 - No reading apparatus
- Kit with trays, M4H medium and R4H reagent
- Divisible tray (TWIN 1C V2)
- Long shelf-life

Reliable

- Proven standardization
- Clear visual reading

Performance

- Update of the molecules tested according to the medical direction and recommendations of the CA-SFM (French Society for Microbiology)
- 97.3% correlation when compared with other reference methods

Reagents

URIFAST® A B G® V2 tray: Tray of 20 wells for 1 test.

Dehydrated antibiotics at their critical concentrations

URIFAST® TWIN 1C V2 tray: Divisible tray (2x10 wells) for 2 tests

Dehydrated antibiotics at their lower critical concentration

M4H medium: Vial of modified Mueller-Hinton medium containing bromothymol blue

R4H reagent: Additive to be used when testing *Proteus mirabilis* strains

Accessory

PRESTO ABG®: Single use inoculator that provides a standardized inoculum

Box of 400 units - Ref. 22204

Catalogue

URIFAST® A B G® V2: Kit of 50 tests - Ref. 22214




URIFAST® TWIN 1C V2: Kit of 50 tests - Ref. 22296

METHODOLOGY

1. Inoculation


Pass the PRESTO ABG® right through the colony and the agar

Prick vertically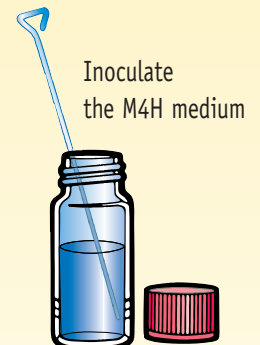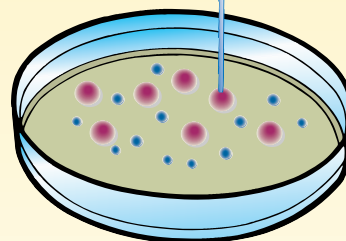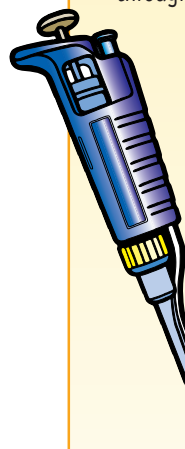


2. Distribution into the wells

100 µl per well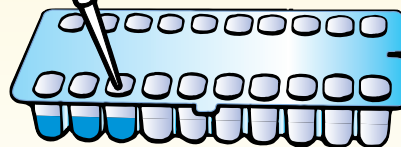

3. Incubation

4.5 hours at 35-37°C

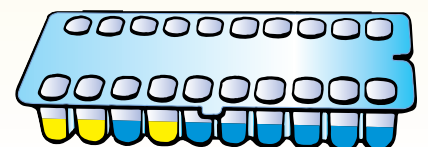4. Reading

Yellow or green: growth
Blue: absence of growth

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ELITech FRANCE SAS